

# Full Arch Case Protocol (Analog)

1. First Appointment
  - ☐ A. Take Preliminary Impressions
  - ☐ B. Take Pre-Op Photos
  - ☐ C. Complete Tooth Shade Guide and Patient Questionnaire
  - ☐ D. Contact Implant Representative to ensure proper multi-unit abutments are ordered and ready for the second appointment.
2. Second Appointment
  - ☐ A. Insert multi-unit abutments
  - ☐ B. Seat and lute together the implant verification jig.
  - ☐ C. Take an x-ray to ensure verification jig is fully seated.
  - ☐ D. Take final impression with custom tray
  - ☐ E. Try-in wax rim, adjust accordingly
  - ☐ F. Take rigid bite registration with the wax rim in place
3. Third Appointment
  - ☐ A. Try-in wax set-up
  - ☐ B. Use Integrity Gingiva Shade Guide to select gingiva shade (take photos if needed and upload to [fullarch@integritydentalservice.com](mailto:fullarch@integritydentalservice.com)).
4. Fourth Appointment
  - ☐ A. Try-in PMMA provisional
  - ☐ B. Let patient wear for 2-3 weeks to confirm occlusion.

Any Questions feel free to contact us:

Phone: (470)-222-2902

Email: [fullarch@integritydentalservice.com](mailto:fullarch@integritydentalservice.com)